




Distal Lateral Femoral Right Jig 00-2360-090-01

Assembly/Disassembly
Instructions

Distal Lateral
Femoral Jig

1

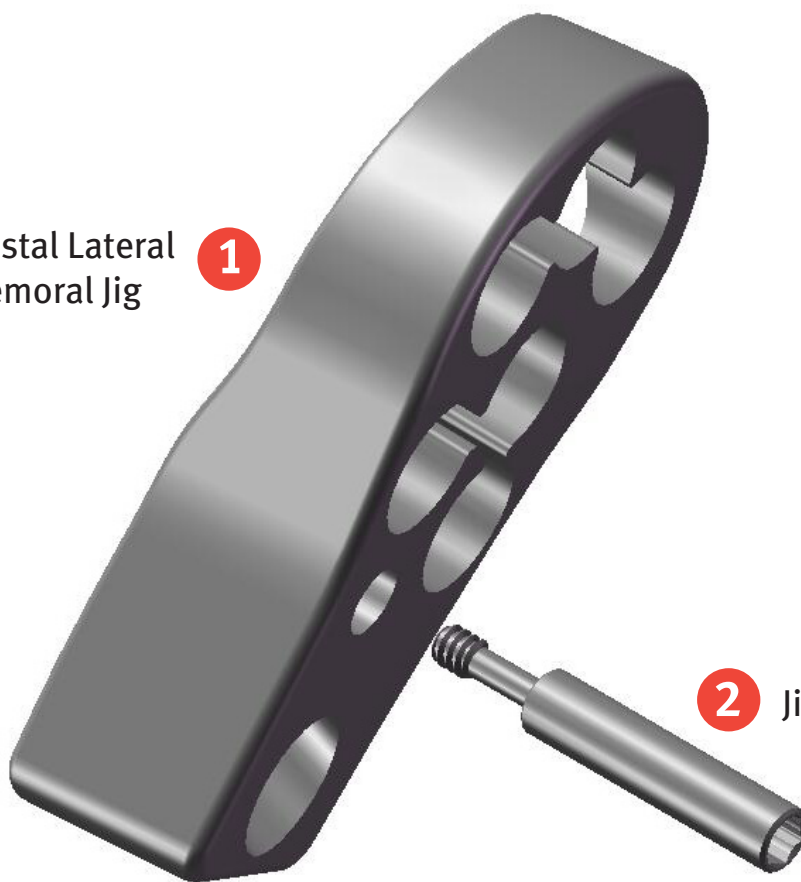

2

Jig Set Screw

Disassembly Instructions for Distal Lateral Femoral Right Jig

1. Unthread the Jig Set Screw **2** out of the Distal Lateral Femoral Jig **1**.
2. End of disassembly.

Assembly Instructions for Distal Lateral Femoral Right Jig

1. Orient the flat surface of the Distal Lateral Femoral Jig **1** facing the Jig Set Screw **2** threads.
2. Thread the Jig Set Screw **2** into the Distal Lateral Femoral Jig **1** fully until it moves freely inside the Jig.
2. End of assembly.



+H124970000509001/\$100610F10R

97-0000-509-00 1003-T22 Printed in USA ©2010 Zimmer, Inc.



zimmer | trauma