




Proximal Lateral Tibial Right Jig (5.5mm)

00-2360-091-01

Assembly/Disassembly
Instructions

Proximal Lateral
Tibial Jig

1

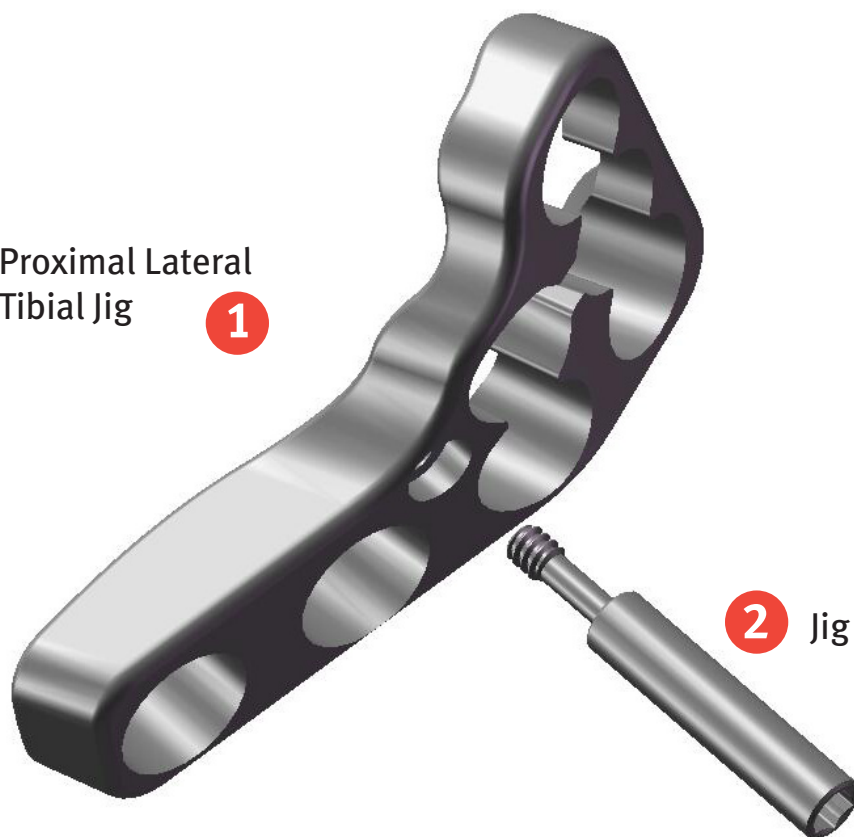

2

Jig Set Screw

Disassembly Instructions for Proximal Lateral Tibial Right Jig

1. Unthread the Jig Set Screw **2** out of the Proximal Lateral Tibial Jig **1**.
2. End of disassembly.

Assembly Instructions for Proximal Lateral Tibial Right Jig

1. Orient the flat surface of the Proximal Lateral Tibial Jig **1** facing the Jig Set Screw **2** threads.
2. Thread the Jig Set Screw **2** into the Proximal Lateral Tibial Jig **1** fully until it moves freely inside the jig.
2. End of assembly.



+H124970000511001/\$100610F10K

97-0000-511-00 1003-T21 Printed in USA ©2010 Zimmer, Inc.



zimmer | trauma