




Proximal Lateral Tibial Left Jig (5.5mm)

00-2360-091-02

Assembly/Disassembly
Instructions

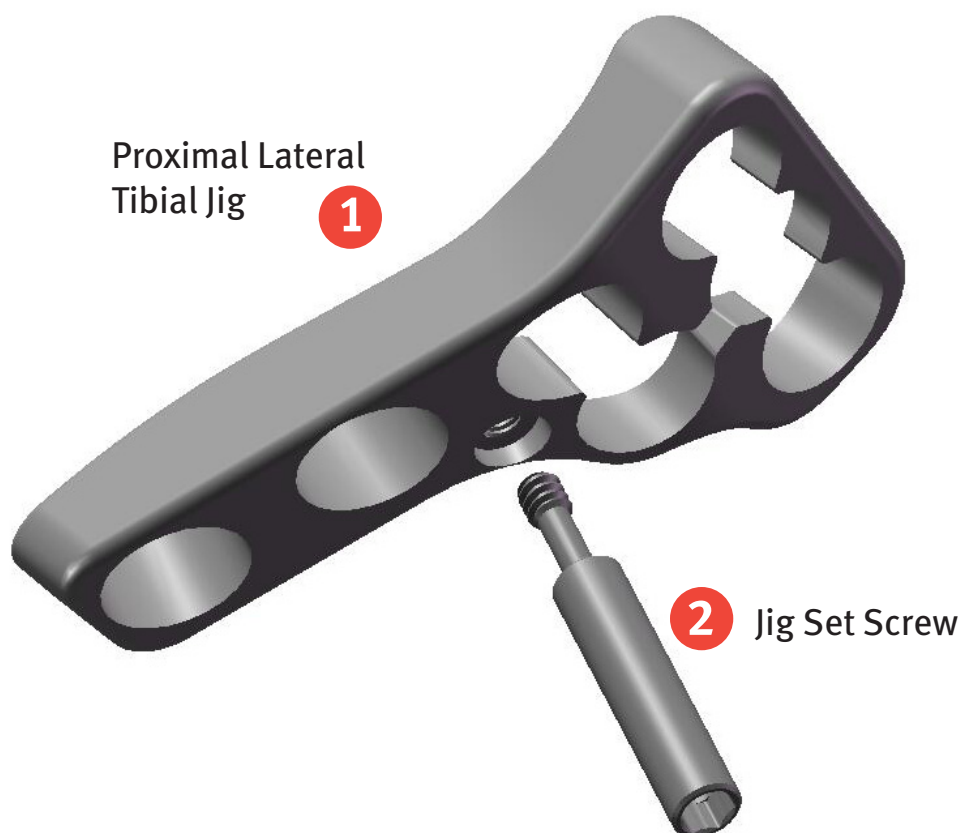

Proximal Lateral
Tibial Jig

1

Jig Set Screw

Disassembly Instructions for Proximal Lateral Tibial Left Jig

1. Unthread the Jig Set Screw **2** out of the Proximal Lateral Tibial Jig **1**.
2. End of disassembly.

Assembly Instructions for Proximal Lateral Tibial Left Jig

1. Orient the flat surface of the Proximal Lateral Tibial Jig **1** facing the Jig Set Screw **2** threads.
2. Thread the Jig Set Screw **2** into the Proximal Lateral Tibial Jig **1** fully until it moves freely inside the jig.
2. End of assembly.



+H124970000512001/\$100610F10L

97-0000-512-00 1003-T24 Printed in USA ©2010 Zimmer, Inc.